

PHILIPS HS1 DEFIBRILLATOR

Get Started Guide



Congratulations you are minutes away from having a life-saving Defibrillator visible and ready to respond at your location.

Your Defibrillator contents should include:

- ✓ HeartStart Quick Setup Guide booklet
- ✓ Mini folder for contents
- ✓ Defibrillator case
- ✓ Defibrillator
- ✓ Emergency Response Plan
- ✓ A4 Sticker & A6 sticker

How to set up your Defibrillator.



Step 1



You must activate your Defibrillator.

To do this, you will need to remove the green tag to start AED self-test. Listen for the “Beep!”



Step 2



When instructed, press Shock button firmly.



Step 3



Wait one minute, and listen for the voice prompt “Ready for use”.



Step 4

Look at the top right hand corner for the blinking green ready light.

This indicates a self-test has passed and your Defibrillator is now active and ready to respond in an emergency.

Step 5



Install the Defibrillator in a location that is clearly visible, accessible and secure.

Step 6



Always store your Defibrillator in its protective case.

For best visibility of Defibrillator location, install directional/AED signage throughout your area.

Step 7



Register your Defibrillator, it is easy and only takes a few minutes. Visit ambulance.vic.gov.au/register, or call 1800 AED REG (233 734) during business hours.

When your Defibrillator is registered, information about its location is made available through Ambulance Victoria's online AED registry. This means more people will be aware of where it's located, in case of an emergency nearby.

Step 8

Maintaining your Defibrillator is important.

Please refer to the HeartStart Quick Setup Guide booklet for further information.



For everything First Aid

1300 360 455

www.stjohnvic.com.au

Training • Supplies • Defibs • Events • Transport Services